

My hearing aid information

Product: ReSound LiNX²

Model: MIH

Serial number (R):

Serial number (L):

Battery size: 312 or 13



Quick guide

ReSound LiNX²™ MIH

Programs:

(if set up by your hearing care professional)

| PROGRAM | WHEN TO USE | BEEPS |
|---------|-------------|-------|
| 1 | | 1 |
| 2 | | 2 |
| 3 | | 3 |
| 4 | | 4 |


Volume control (C):

(if set up by your hearing care professional)

| ACTION | RESULT |
|--|------------------|
| Rotate forward (toward your face) | Volume increases |
| Rotate backward (away from your face) | Volume decreases |

Get to know your hearing aid

- A. Microphone
- B. Battery door
- C. Program button
- D. Retrieval line
- E. Vent

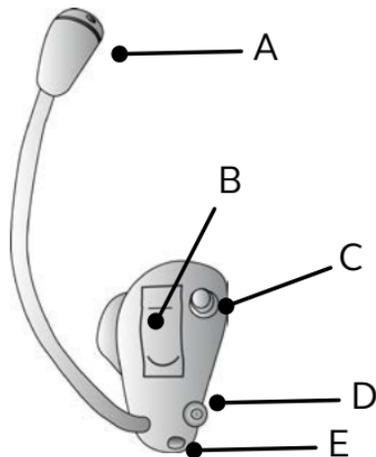


View ReSound hearing aid tutorial video

F. Wax guard

G. Manufacturer

H. Serial number

I. Model

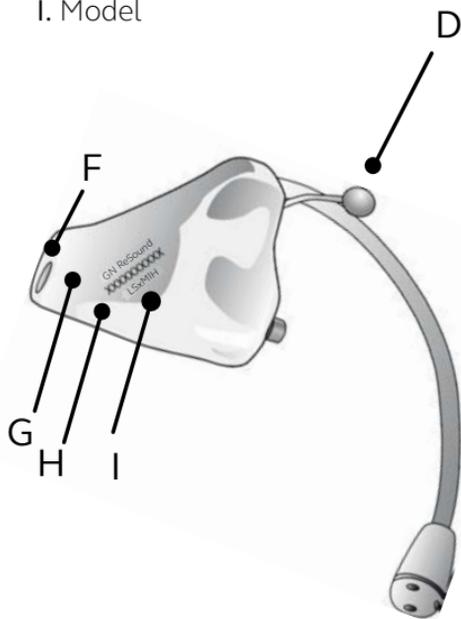


What you need to know

Getting started: On/Off function

ON: Insert battery & completely close battery door (B).


Program 1 will be active.

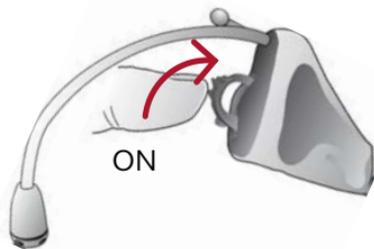

OFF: Open the battery door (B).

Use

fingertip to pull it all the way.

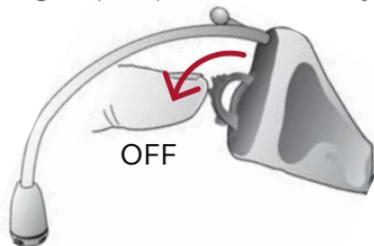

Inserting/Replacing the battery:

1. Open the battery door (B) completely by using your fingertip.



2. Remove protective sticker from battery and wait 2 minutes before inserting the battery into the hearing aid to allow full activation of the battery.



3. Insert the new battery with the positive side (flat side) in the correct position, as shown.




4. Gently close the battery door (B). If it doesn't close easily, check to see if the battery is upside down.

Regular care & maintenance

Daily: Clean your hearing aid.

Keep your hearing aids dry. Clean the hearing aids using a soft cloth or tissue.

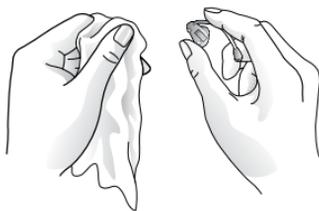

Monthly: Change the wax guard (F) or see your hearing care professional.

For changing CeruSTOP™ (white) wax guards, complete the following steps:


Removal: Insert the removal side of the wax guard tool into the used wax guard so that the shaft of the tool is touching the rim of the wax guard. Slowly pull the wax guard straight out.




Insertion: To insert the new wax guard, gently press the replacement side of the wax guard tool straight into the hole of the sound outlet until the outer ring lies flush with the outside of the receiver. Pull the tool straight out. The new wax guard will remain in place.

For changing HF3 wax guard (F), complete the following steps:

1. Brush the sound outlet area with the sound outlet pointed down.

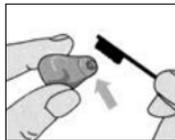


2. Insert the threaded end of the wax guard tool into the used wax guard, and rotate clockwise.

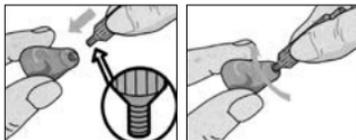

3. Gently pull until the used guard is removed.




4. Discard the used guard in the slot located

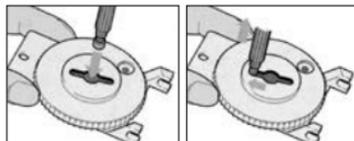


in the wax guard kit by pressing it into the center, sliding it to one end of the slot, and pull until the filter is discarded.


5. Flip the wax guard tool around, locate a new guard in the dial, and press the tip of the tool into the center of the dial.

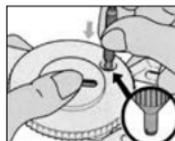

6. Gently pull the new filter out of the dial.

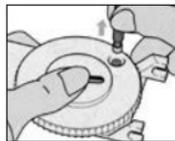

7. Press the new guard to the canal tip of the hearing aid. Move the tool gently side to side and pull it away from the hearing aid.

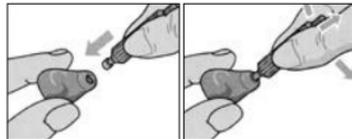

Troubleshooting

| SYMPTOM | CAUSE | POSSIBLE REMEDY |
|------------------------------|-----------------------------------|---|
| No sound | Battery door not closed | Turn on by closing the battery door (B) |
| | Dead battery | Replace battery |
| | Battery door will not close | Insert battery properly |
| | Blocked wax guard | Replace wax guard (F) or consult your hearing care professional |
| Not loud enough | Blocked sound outlet | Change wax guard (F) or consult your hearing professional |
| | Change in hearing, volume too low | Consult your hearing care professional |
| | Excessive ear wax | Consult your hearing care professional |
| Excessive whistling/feedback | Incorrect placement in ear | Re-insert hearing aid carefully |
| | Excessive ear wax | Clean guard/consult your hearing care professional |
| | If none of the above | Consult your hearing care professional |

If there are any other problems not mentioned in this guide, please contact ReSound or your hearing care professional.

View ReSound hearing aid tutorial videos at youtube.com/resoundus.

For more information

Refer to the user guide included in your hearing aid box.

Your hearing care professional can be reached at:

View ReSound hearing aid tutorial videos at youtube.com/resoundus.

Manufacturer according to FDA

ReSound North America

8001 E. Bloomington Freeway
Bloomington, MN 55420
1.888.735.4327
resound.com

Manufacturer according to Health Canada

ReSound Canada

303 Supertest Road
Toronto, Ontario M3J 2M4
1.888.737.6863
resound.com

ReSound Government Services

8001 E. Bloomington Freeway
Bloomington, MN 55420
1.800.392.9932
www.resound.com/veterans



400560011

400560011 Rev E 2016.11