

My hearing aid information

Product: ReSound LiNX²

Model: 61

Serial number (R):

Serial number (L):

Battery size: 312



ReSound GN

Quick guide

ReSound LiNX²™ 61


Programs:

(if set up by your hearing care professional)

PROGRAM	WHEN TO USE	BEEPS
1		1
2		2
3		3
4		4

Get to know your hearing aid

- A. Program button
- B. Microphones
- C. Receiver
- D. Wax guard
- E. Dome
- F. Battery door
- G. Right/Left indicator

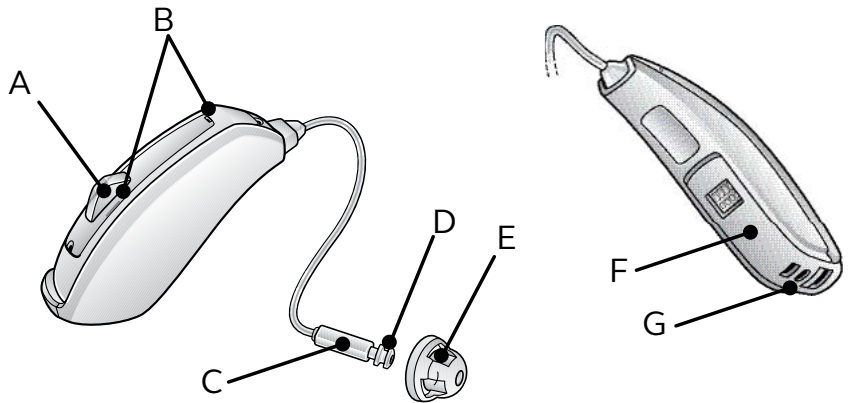


View ReSound's hearing aid tutorial videos at [youtube.com/resoundus](https://www.youtube.com/resoundus).


What you need to know

Getting started: On/Off function

ON: Insert battery & completely close battery door (F). Program 1 will be active.

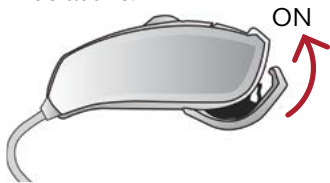


OFF: Open the battery door (F). Use fingertip to pull it all the way.

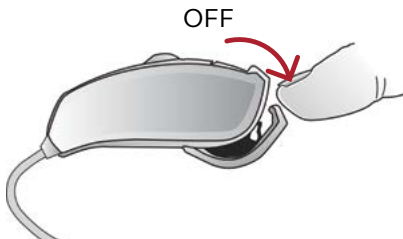

Regular care & maintenance

Daily: Clean your hearing aid.

Keep your hearing aids dry. Clean the hearing aids using a soft cloth or tissue.

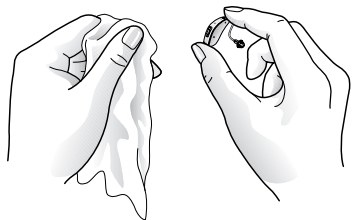

Weekly: Clean the receiver tubes and domes (E).

Use a damp cloth to clean the receiver tube and dome on the outside. Do not use water when you are cleaning the receiver tubes.

Inserting/Replacing the battery:

1. Open the battery door (F) completely by using your fingertip.



2. Remove protective sticker from battery and wait 2 minutes before inserting the battery into the hearing aid to allow full activation of the battery.



3. Insert the new battery with the positive side (flat side) in the correct position, as shown.



4. Gently close the battery door. If it doesn't close easily, check to see if the battery is upside down.

Flight mode:

Some airlines prohibit the use of wireless hearing aids (RF transmitters) onboard. If required by your airline, enable flight mode:

Disable wireless operation by opening and closing the battery compartment three times within ten seconds (open-close, open-close, open-close). Your instruments will now be in flight mode.

The hearing aids must function in flight mode for 10 seconds before wireless operation can be re-enabled.


To re-enable wireless operation, open and close the battery door one time and wait thirty seconds before opening the battery door again to avoid accidentally entering flight mode again.

Regular care & maintenance

As needed: Change dome (E)

ReSound standard domes


1. Push the new dome over the receiver.
2. Make sure that the new dome is properly and securely maintained.

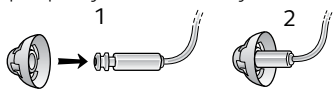

ReSound tulip domes

Tulip domes consist of two “petals”. It is important that the largest petal is the outermost petal. To ensure this:


1. Push the largest petal away from the receiver tube using a finger. This bends the petal forward.
2. Then push the largest “petal” back, and it will be placed on top of the smaller petal.

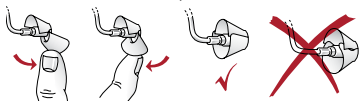

Monthly: Change the wax guard or see your hearing care professional.

For changing CeruSTOP™ (white) wax guards, complete the following steps:

Removal: Insert the removal side of the wax guard tool into the used wax guard so that the shaft of the tool is touching the rim of the wax guard. Slowly pull the wax guard straight out.

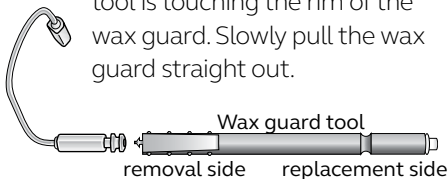

Insertion: To insert the new wax guard, gently press the replacement side of the wax guard tool straight into the hole of the sound outlet until the outer ring lies flush with the outside of the receiver. Pull the tool straight out. The new wax guard will remain in place.

Beep/tone indicators

BEEP/TONE	INDICATION	ACTION REQUIRED
Ding, ding, ding, dong	Low battery	Change battery
Ding (5 or 10)	Hearing aid is starting up	After dings cease, hearing aid is turned on
Ding-ding, ding-ding, etc.	Hearing aid is in flight mode	Use in flight mode if desired, or open-close battery door (F) to exit flight mode
Ripple tone	Hearing aid is in streaming mode	Use with wireless accessory or press multi-function button to exit streaming mode

View ReSound hearing aid tutorial videos at youtube.com/resoundus.

Troubleshooting

SYMPTOM	CAUSE	POSSIBLE REMEDY
No sound	Battery door (F) not closed	Turn on by closing the battery door (F)
	Dead battery	Replace battery
	Battery door (F) will not close	Insert battery properly
	Blocked earmold or tube	Clean earmold or tube
	Blocked wax guard	Replace wax guard (D) or consult your hearing care professional
Not loud enough	Blocked earmold or dome (E)	Clean earmold, replace dome (E), replace guard (D)
	Change in hearing, volume too low	Consult your hearing care professional
	Excessive ear wax	Consult your hearing care professional
Excessive whistling/feedback	Incorrect earmold placement	Re-insert earmold carefully
	Incorrect dome (E) placement	Re-insert dome (E)
	Excessive ear wax	Clean guard/consult your hearing care professional
	If none of the above	Consult your hearing care professional
Wireless does not work	Device may be in flight mode	Open and close the battery compartment (F) once. Wireless operation will reactivate ten seconds later.

If there are any other problems not mentioned in this guide, please contact ReSound or your hearing care professional.

For more information

Refer to the user guide included in your hearing aid box.

Your hearing care professional can be reached at:

View ReSound hearing aid tutorial videos at youtube.com/resoundus.

Manufacturer according to FDA

ReSound North America

8001 E. Bloomington Freeway
Bloomington, MN 55420
1.888.735.4327
resound.com

Manufacturer according to Health Canada

ReSound Canada

303 Supertest Road
Toronto, Ontario M3J 2M4
1.888.737.6863
resound.com

ReSound Government Services

8001 E. Bloomington Freeway
Bloomington, MN 55420
1.800.392.9932
www.resound.com/veterans



400568011

400568011 Rev E 2016.11